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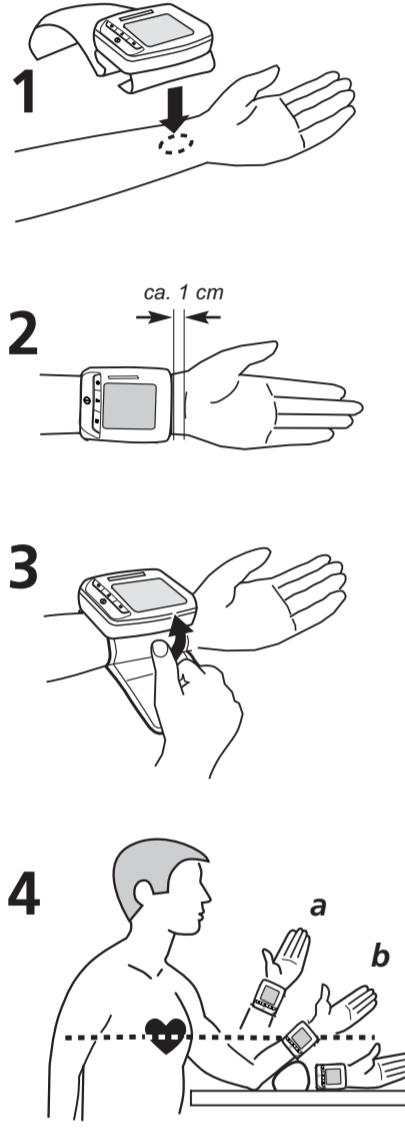
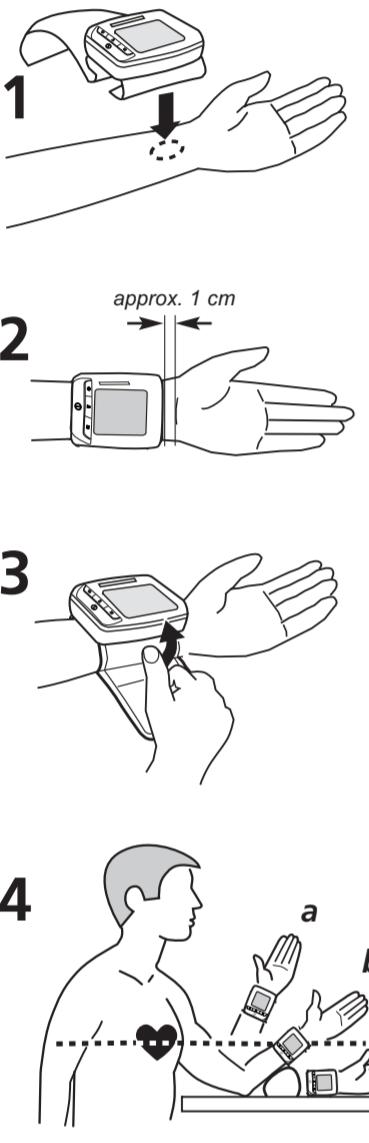
<p


GB Blood Pressure Monitor BW 310

Instruction Manual Please read carefully!

GB Unit and LC-Display

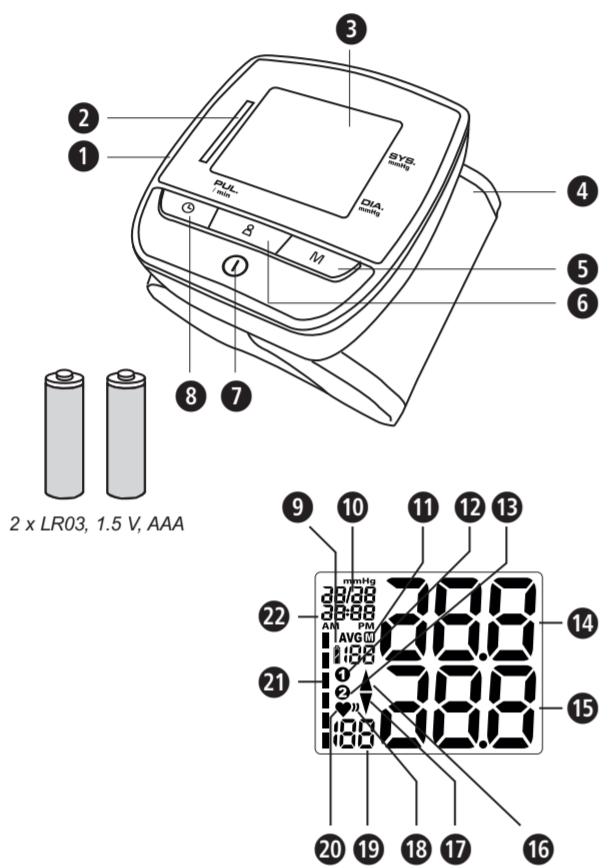Proper use

- This fully automatic electronic blood pressure monitor is intended for measuring the blood pressure at home. It is a non-invasive blood pressure measurement system to measure the diastolic and systolic blood pressure and pulse of an adult using a non-invasive technique by means of a cuff, which needs to be fitted on the wrist.

Contraindications

- The device is not suitable for measuring the blood pressure of children. Ask your doctor before using it on older children.
- Persons who suffer from arrhythmia, diabetes, cardiovascular problems or who have had a stroke should consult their doctor before using the device.

Common factors of wrong measurements

- Please keep quiet for 5-10 minutes and avoid eating, drinking alcohol, smoking, exercising and bathing before taking a measurement. All these factors will influence the measurement result.
- Remove watches or other spruce from your wrist before measurement.
- Always measure on the same wrist (normally left).
- Measurements should be taken regularly at the same time of each day, as the blood pressure varies even during the day.
- Any effort to support the arm during measurement may increase the measured blood pressure.
- Make sure, you are in a comfortable, relaxed position and do not move or constrict your muscles during measurement. Use a cushion to support your arm if necessary.
- If the wrist is not at heart level, a false reading may be obtained.
- A loose or open cuff causes false readings.
- With repeated measurements, blood accumulates in the wrist, what may lead to false readings.

- Consecutive blood pressure measurements should be carried out after a 5-minute pause or after the arm has been held up in order to allow the accumulated blood to flow normally.

WARNING

These warning notes must be observed to prevent any injury to the user.

CAUTION

These notes must be observed to prevent any damage to the device.

NOTE

These notes give you useful additional information on the installation or operation.

Device classification: type BF applied part

LOT

LOT number

Manufacturer

Date of manufacture

GB Safety Instructions

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.

- The unit is intended only for use in the home. Consult your doctor prior to using the blood pressure monitor if you are concerned about health matters.
- Only use the unit for its intended purpose in accordance with the instructions for use. Warranty claims become void if the unit is misused.
- Please consult your doctor prior to using the unit if you suffer from illnesses such as arterial occlusive disease.
- The unit may not be used to check the heart rate of a pacemaker.

- Like on any other oscillometric blood pressure measurement devices, certain medical conditions can affect the measurement accuracy, among others: cardiac arrhythmias, low blood pressure, circulatory disorders, states of shock, diabetes, pregnancy, pre-eclampsia etc. Always consult a physician before you use the device.
- This device is not suitable to provide a medical diagnosis. The measurement results are for reference only. Contact your physician to get information about treatment and medicine, which are necessary for you.

- The arrhythmia function does not replace a professional cardiac examination. Check with your doctor if any treatment or medical action is necessary for you.
- A heartbeat that varies by 25% or more from the average of all heartbeats during a measurement may cause difficulties to determine the correct blood pressure.

- The "Blood pressure classification WHO" is not suitable to replace a medical diagnosis. Ask your physician to individually evaluate your blood pressure.
- If you experience discomfort during a measurement, such as pain in the wrist or other complaints or in case the device does not stop to inflate the cuff, press the **1**-button **7** to release the air immediately from the cuff. Loosen the cuff and remove it from your wrist.

- Blood pressure measurements in high repetition rates may lead to unwanted side effects, e.g. nerve compression or blood clots.
- Blood pressure measurements - especially if executed frequently - may lead to temporary marks on the skin. These marks may be visible for several days in individual cases. If necessary, contact your physician to get more information.

- This device is not designed to be used by persons with limited physical, sensory or mental abilities, or by persons with insufficient experience and/or knowledge, unless under observation by a person responsible for their safety, or unless they have been instructed in the use of the device.
- This device is intended to be used by adults. The unit is unsuitable for children or babies. Contact your doctor, if you want to use this device on young persons.

- Do not allow children to use the unit. Medical products are not toys!
- The swallowing of small parts like packaging bag, battery, battery cover and so on may cause suffocation.

- The user needs to check that the equipment functions properly and that it is in proper working condition before using it.
- The device needs to be used in accordance to the specified ambient conditions (see "Technical specifications") only. Otherwise the accuracy of readings might be affected.

- When taking readings, remember that the daily values are influenced by several factors. Smoking, consumption of alcohol, drugs and physical exertion influence the measured values in various ways.
- Measure your blood pressure before meals.
- Before taking readings, allow yourself at least 5-10 minutes rest.

- If the systolic and diastolic readings seem unusual (too high or too low) on several occasions, despite correct use of the unit, please inform your doctor. This also applies to the rare occasions when an irregular or very weak pulse prevents you from taking readings.

- Do not use the equipment where flammable gas (such as anaesthetic gas, oxygen or hydrogen) or flammable liquids (such as alcohol) are present.

- Do not take measurements, if any other equipment is applied to the same limb for monitoring. This could lead to disturbances or failures.

- After a lymph node removal operation a physician should be consulted, before a blood pressure measurement is executed with this device.

- Never apply the cuff over hurt skin or in case of an inserted catheter.

- Do not apply the cuff on any other part of the body than the left wrist.

- Before applying the cuff, make sure there are no wrinkles in the cuff, as these could cause bruises.

- The cuff size should be chosen according to the wrist circumference. Non fitting cuffs may lead to inaccurate measurements.

- The cuff must not be applied over clothes as this may lead to injuries or inaccurate readings.

- Do not kink or fold the cuff and do not pull it over sharp edges.

- No modification of this unit/device is allowed.

- Please do not attempt to repair the unit yourself in the event of malfunctions, since this will void all warranty claims. Only have repairs carried out by authorised service centres.

- Protect the unit against moisture. If liquid does manage to penetrate into the unit, remove the batteries immediately and do not continue to use the unit. In this case contact your specialist dealer or notify us directly.

- Extreme temperatures, humidity or altitudes may impair the performance of the device.

- Please do not use a dilution agent, alcohol or petrol to clean the unit.

- Do not hit the device heavily or let it fall down from a high place.

- Do not insert any objects into the device opening and never push the buttons with heavy force or with the help of peak objects. This may lead to damages of the device.

- If the unit is not going to be used for a long period, please remove the batteries.

SAFETY NOTES FOR BATTERIES

- Do not disassemble batteries!
- Replace all batteries if the unit display shows the low battery symbol.

- Never leave any low battery in the battery compartment since it may leak and cause damage to the unit.

- Increased risk of leakage! Avoid contact with skin, eyes and mucous membranes! If battery acid comes in contact with any of these parts, rinse the affected area with copious amounts of fresh water and seek medical attention immediately!

- If a battery has been swallowed, seek medical attention immediately!

- Replace all of the batteries simultaneously!

- Only replace with batteries of the same type, never use different types of batteries together or used batteries with new ones!

- Insert the batteries correctly, observing the polarity!

- Keep batteries out of children's reach!

- Do not attempt to recharge batteries! There is a danger of explosion!

- Do not short circuit! There is a danger of explosion!

- Do not throw into a fire! There is a danger of explosion!

- Do not throw used batteries into the household refuse; put them in a hazardous waste container or take them to a battery collection point, at the shop where they were purchased!

Unit and LCD display

- | | |
|--|--|
| ① Lid of the Battery Compartment | ② Blood Pressure Indicator (green - yellow - orange - red) |
| ③ LC-display | ④ Wrist cuff |
| ⑤ M - button (Memory Recall) | ⑥ A - button (User Setting) |
| ⑦ O - button (START/STOP) | ⑧ C - button (Clock/Date) |
| ⑨ Change Battery symbol | ⑩ Average value |
| ⑪ User memory 1 | ⑫ User memory 2 |
| ⑬ Display of Systolic Pressure | ⑭ Inflation symbol |
| ⑮ Display of Diastolic Pressure | ⑯ Deflation symbol |
| ⑰ Arrhythmia display (arrhythmia cordis) | ⑲ Display of Pulse Rate |
| ⑳ Pulse symbol | ⑳ Blood Pressure Indicator |
| ⑳ Pulse symbol | ⑳ Display of Time / Date |

What is blood pressure?

Blood pressure is the pressure created in the blood vessels by each heart beat. When the heart contracts (= systole) and pumps blood into the arteries, it brings about a rise in pressure. The highest value is known as the systolic pressure and is the first value to be measured. When the cardiac muscle relaxes to take in fresh blood, the pressure in the arteries also falls. Once the arteries are relaxed, a second reading is taken – the diastolic pressure.

How is blood pressure measured?

The MEDISANA BW 310 is a blood pressure unit which is used to measure blood pressure from the wrist. The measurement is carried out by a microprocessor, which, via a pressure sensor, measures the vibrations resulting from the inflation and deflation of the cuff over the artery.

Blood pressure classification WHO

These values have been specified by the World Health Organisation (WHO) without making any allowance for age.

Low blood pressure	systolic < 100	diastolic < 60
Normal blood pressure	(green display area ② / ④)	(green display area ② / ④)

systolic 100 - 139 diastolic 60 - 89

Forms of high blood pressure / hypertension

Mild hypertension (yellow display area ② / ④)

systolic 140 - 159 diastolic 90 - 99

(orange display area ② / ④)

systolic 160 - 179 diastolic 100 - 109

(red display area ② / ④)

systolic ≥ 180 diastolic ≥ 110

WARNING
Blood pressure that is too low represents just as great a health risk as blood pressure that is too high! Fits of dizziness may lead to dangerous situations arising (e.g. on stairs or in traffic).

Influencing and evaluating readings

- Measure your blood pressure several times, then record and compare the results. Do not draw any conclusions from a single reading.
- Your blood pressure readings should always be evaluated by a doctor who is also familiar with your personal medical history. When using the unit regularly and recording the values for your doctor, you should visit the doctor from time to time to keep him updated.
- When taking readings, remember that the daily values are influenced by several factors. Smoking, consumption of alcohol, drugs and physical exertion influence the measured values in various ways.
- Measure your blood pressure before meals.
- Before taking readings, allow yourself at least 5-10 minutes rest.
- If the systolic and diastolic readings seem unusual (too high or too low) on several occasions, despite correct use of the unit, please inform your doctor. This also applies to the rare occasions when an irregular or very weak pulse prevents you from taking readings.

Starting up

Insert / replace batteries

You must insert the batteries provided before you can use your unit. The lid of the battery compartment ① is located on the left hand side of the unit. Open it by pushing the cover outward and remove it. Insert the 2 AAA LR03 type 1.5 V batteries supplied. Ensure correct polarity when inserting the batteries (as marked on the battery compartment cover).

Close the battery compartment. Replace the batteries immediately, if the partial change battery symbol ⑨ ⑩ flashes on the display ③ and "E6" is appearing at the same time or if nothing appears on the display after the unit has been switched on. If the partial change battery symbol ⑨ ⑩ appears on the display constantly together with "E6", the batteries are already in poor condition. If the full change battery symbol ⑨ ⑩ appears, the batteries are in good condition.

Settings

After inserting the batteries, time and date need to be adjusted correctly. The device automatically switches to settings mode after insertion of the batteries. If you want to get to settings mode manually, press and hold the ④ - button ⑤ when power off to power on the device. To call up the measured values stored, press the ④ - button ⑤ to select the user (① or ②) and then the M - button ⑤. The average values of the last 3 measurements appear ⑩ on the display, together with "AVG". Pressing the M - button ⑤ again, the last measurement value will be shown. Repeatedly pressing the M - button ⑤ displays the respective values measured previously. If you have reached the final entry and do not press a button, the unit automatically switches itself off after approximately 2 min. in memory recall mode. Memory recall mode can be exited at any time by pressing the ④ - button ⑤ which will also switch the unit off. The oldest value is deleted if 60 measured values have been stored in the memory and a new value is saved.

Delete saved values

If it is necessary to interrupt a blood pressure measurement for any reason (e.g. the patient feels unwell) the ④ - button ⑤ can be pressed at any time. The device immediately decreases the cuff pressure automatically.

Display stored values

This unit features 2 separate memories, each with a capacity of 60 memory slots. Results are automatically stored in the memory selected by the user. Press the ④ - button ⑤ when power off to power on the device. To call up the measured values stored, press the ④ - button ⑤ to select the user (① or ②) and then the M - button ⑤ to select the user (① or ②). The average values of the last 3 measurements appear ⑩ on the display, together with "AVG". Pressing the M - button ⑤ again, the last measurement value will be shown. Repeatedly pressing the M - button ⑤ displays the respective values measured previously. If you have reached the final entry and do not press a button, the unit automatically switches itself off after approximately 2 min. in memory recall mode. Memory recall mode can be exited at any time by pressing the ④ - button ⑤ which will also switch the unit off. The oldest value is deleted if 60 measured values have been stored in the memory and a new value is saved.

Error messages and error remedying

Error indicators

The following symbol will appear on the display when measuring abnormal:

Symbol	Cause	Correction

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